

# How the Chair-Check® II System Can Work for You

Chair-Check II is a system utilized to aid in the monitoring of fall risk patients, allowing the nursing and aide staff of either long-term or short-term care facilities to determine if a patient is still seated in a chair, wheelchair, or on a toilet seat.

The Chair-Check II system consists of a control unit and a disposable pressure-sensitive Sensormat®. When the patient's weight on the Sensormat is removed for a pre-selected number of seconds, the control unit emits an alarm tone to let the staff know that the patient is no longer on the Sensormat and is in an unsafe position.

## **How to Use the Chair-Check® II System**

**WARNING: Use of mats not manufactured by Bed-Check Corporation will impair the effectiveness of the Chair-Check II unit.**

1. Plug the Chair-Check II Sensormat cord connector into the bottom of the Chair-Check II control unit.
2. Attach the control unit to the back of the chair out of patient's sight and reach, but where it is readily accessible to nursing personnel.
3. Place Sensormat in the seat of the chair so that it will be beneath the patients buttocks. Cover the Sensormat with a foam cushion or incontinence pad, if desired.
  - a. For Potty-Check Sensormat, see the inservice flyer #937
4. Seat patient in chair. The control unit will arm itself as indicated by a beep and flashing red light next to the "Reset" button.