

## **Care and Maintenance**

The control unit can be wiped with a damp cloth or sponge using mild detergents or sanitizing solution and water. Never use alcohol, strong detergents or harsh chemical cleaners. **NEVER IMMERSE THE CONTROL UNIT IN ANY LIQUID. AVOID UNIT PLACEMENT IN ANY AREA WHERE IT MIGHT COME IN CONTACT WITH BODILY FLUIDS. DAMAGE CAUSED BY ANY FLUID IS NOT UNDER WARRANTY.**

- ✓ Chair-Check II control unit may not be effective with another manufacturer's sensing device. Use of another sensing device may cause damage to the Chair-Check II control unit and will void the Chair-Check II warranty.
- ✓ Bending or folding the Chair-Check II Sensormat may cause loss of effective operation.
- ✓ Internal maintenance should only be performed in a static-controlled environment by qualified personnel.

The Sensormat can be cleaned in a similar manner and with similar solutions used to clean vinyl mattress covers.

To eliminate the possibility of cross contamination, the Sensormat should only be used for one patient and then discarded.

Use beyond the warranty period can cause breakdown of the internal conductive ink and render the Sensormat inoperative.