

Battery

The Chair-Check II utilizes a standard alkaline 9-volt battery or optional rechargeable battery. The more the alarm actually sounds the less service you will get from a battery. If a low volume constant series of short beeps (similar to the low battery tones of a smoke alarm) occurs even when the patient is seated on the mat, the battery is low and should be replaced. Test the system daily for effectiveness.

NOTE: The Chair-Check II control unit includes an alkaline battery. **DO NOT ATTEMPT TO CHARGE AN ALKALINE BATTERY.** Charging an alkaline battery will damage the control unit and void the warranty. The charging jack available on the control unit is to be used for charging Nickel-metal Hydride batteries only. The Nickel-metal Hydride battery and recharger are optional items and must be special ordered. Nickel-Cadmium batteries are not recommended.