



Plug the Control Unit's external power supply cord into the Control Unit (Port designated as "Power") first. Then plug the power supply into an AC wall outlet. IF THE SENSORMAT IS PLUGGED IN AND WEIGHT IS APPLIED TO THE SENSOR-MAT, THE CONTROL UNIT WILL ALARM IF THE NURSE CALL CORD IS NOT PLUGGED INTO THE UNIT WHEN POWER IS SUPPLIED. The Control Unit will ignore the absence or presence of the nurse call cord if the Relay Action is set to "no pullout protection". (Refer to the Program Guide.)

Connect the Control Unit's nurse call cord (phone style plug end) to the Control Unit (Port designated as "Nurse Call "). Plug the 1/4 inch phono style plug into the nurse call system. Some systems may require an adapter which may be purchased with your Bed-Check System (For Details see "Interfacing the Bed-Check System with Your Nurse Call System").

The Control Unit and cords must be placed in such a location or manner in which it would not subject them to damage from moving bed parts or positioning of the bed. Do not position either component of the system such that it may cause one to pull against the other, causing stress to the connecting wire or its connecting plug.

Once the patient is positioned properly on the Sensormat, the yellow **HOLD** indicator will begin to flash. After a preset monitoring delay (the delay time is factory preset to 5 seconds) the Control Unit will begin monitoring automatically as indicated by the green **MON**itoring indicator. To bypass this delay, press the **RESET** button and immediate monitoring will be indicated by the green **MON** indicator.

ALWAYS PHYSICALLY INSPECT THE STATUS OF THE CONTROL UNIT BY VIEWING THE ILLUMINATED INDICATORS ON THE FACE OF THE CONTROL UNIT BEFORE LEAVING THE PATIENT'S ROOM. IF THE CONTROL UNIT IS IN THE HOLD MODE, PRESS THE RESET BUTTON BEFORE LEAVING THE PATIENT TO IMMEDIATELY BEGIN MONITORING. There is the possibility of the patient exiting the bed or chair during the **HOLD** period without causing the system to alarm.

The Bed-Check Control Unit, its power supply and the Bed-Check Sensormat have been tested and listed by a Nationally Recognized Testing Laboratory as a system, and have met all applicable FDA regulations. The Bed-Check system is categorized as Nurse Call Accessory Equipment and is tested to UL® Standard 1069 by Edison Testing Labs (ETL), and CSA® 22.2 No. 205R.