

BASIC-CHECK™ IN-SERVICE REVIEW

PROPER INSTALLATION OF THE BASIC-CHECK SYSTEM ON A CHAIR WHEN USED WITH A CHAIR SENSORMAT®

(Also consult Basic-Check User Guide)



A. Remove Sensormat from plastic container. Use the adhesive strips on the back of Sensormat to keep it properly positioned and to ensure reliability. Usually the Sensormat should be placed in the seat about 4" from the back of the chair, across the width of the seat.



B. Plug the Chair Sensormat's telephone style jack into the Basic-Check control unit.

IMPORTANT: Use only Sensormats manufactured by Bed-Check Corporation with the Basic-Check control unit.



C. To secure the unit out of reach of the patient/resident, attach the Basic-Check control unit to the back of the chair using either the standard metal hanger or the Velcro® on the back.

Record the date put in use on the Sensormat cord tag.



D. Seat the patient in the chair. The Basic-Check will arm itself and begin monitoring automatically. It will beep one time to indicate monitoring is beginning.



E. When patient/resident attempts to exit the chair or leans too far forward, the alarm will sound immediately. Alarm requires immediate intervention, but will silence and restart monitoring if the patient sits back down.



F. When an alarm has sounded, first assist the patient. The alarm may be silenced by seating the patient on the Sensormat, automatically putting Basic-Check into monitor mode. Otherwise, silence the alarm by pushing the RESET button, putting Basic-Check into standby mode.



G. To remove the patient/resident from the chair, first assist him/her up, briefly allowing the Basic-Check to alarm. Press the RESET button to silence the alarm, putting the Basic-Check into standby mode.

IMPORTANT:

Check for System Effectiveness Daily and/or at shift change.

Always check correct positioning of Sensormat after any alarm.

Use only Bed-Check products to maintain Bed-Check Warranty and to ensure System compatibility.

The physical characteristics, habits and factors that make each patient at risk for falls are different and may change daily. Please reassess as necessary.



Bed-Check
CORPORATION
Reducing Falls Through Innovative
Technology Since 1978

P.O. Box 170, Tulsa, Oklahoma 74101
1-800-523-7956 • 1-918-592-3338
Fax: 1-918-582-9828
www.bedcheck.com