

than 100 lbs. Assure proper operation before putting into service.

8. Do not position either the unit or the Sensormat in such a manner that causes one to pull on the other, causing stress to the connector wire or the plug.

#### **Care and Maintenance**

The control unit can be wiped with a damp cloth or sponge using mild detergents or Sanitizing solutions. Never use alcohol or strong chemical cleaners. **NEVER IMMERSE THE CONTROL UNIT IN ANY LIQUID. AVOID PLACING THE CONTROL UNIT IN ANY AREA WHERE IT MIGHT COME IN CONTACT WITH BODY FLUIDS. DAMAGE BY FLUIDS VOIDS THE WARRANTY.**

- ✓ Internal maintenance should only be performed by qualified personnel.
- ✓ Bending or folding the Sensormat may cause loss of effective operation.

#### **Battery**

**NOTE: The Basic-Check control unit includes an alkaline battery. A low battery is indicated by a low constant beeping, even while the patient is on the Sensormat. DO NOT ATTEMPT TO RECHARGE THE BATTERY.**