



© 2003

## Some things never change. Falls can happen.

The mind can overrun the body. It happens to all of us.  
But for fall risk patients or residents, fall likelihood can be reduced.

Through a quarter century of designing and manufacturing fall and restraint reduction monitoring systems, Bed-Check® Corporation has learned what caregivers want most.

**That doesn't change either:**

**Simplicity. Versatility. Reliability.**

Introducing the **NEW**

## Classic-Check™

With classic features for monitoring fall risk patients and residents:

- NEW** • DAY/NIGHT MODE, with 7 volume settings, programmable to different volumes for Day and Night.
- NEW** • 10 programmable alarm tone settings. 6 tunes and 4 tones.  
**Easily distinguishable from other equipment.**
- NEW** • Night Light – Soft, blue light within splash guard.
  - Nurse call system compatibility.
  - Monitor bed or bedside chair.
  - Variable “delay” settings: 1, 2, or 3 seconds (off Sensormat® until alarm).
  - **No on/off switch to remember - monitors automatically. Helps you meet JCAHO requirements.**
  - Utilizes any Bed-Check Corporation Sensormat.
  - “Hold Mode”, designed to enable removal of patient or resident by staff, without alarm sounding.
  - Monitoring will automatically re-start within 25 seconds when patient is on Sensormat, even if the system has been placed in “hold mode”.
  - Convenient Velcro® mounting strap, or optional wire holster.
  - Designed and tested to withstand high environmental static voltage.
  - Integral 10' power and nurse call system cords.
  - 110 VAC powered (no batteries to replace).
  - Every unit tested before shipping.



Day Mode



Night Mode



All this and the reliability Bed-Check Corporation is known for in products and service.  
Classic features to meet your continuing classic needs for fall and restraint reduction.  
**The new Classic-Check**

**Purpose:** The Classic-Check Control Unit is intended to be used in conjunction with a Sensormat<sup>®</sup> as a system for the monitoring of fall risk patients or residents in a bed, chair or wheelchair. Classic-Check provides an audible alarm to alert the caregiver when the person begins to get up unassisted. Classic-Check is nurse call system compatible, and comes with an integral ten foot cord (standard) for connection if desired.

**Components:** The System consists of the Classic-Check Control Unit with integral power and nurse call system cords, and a pressure sensitive Sensormat (ordered separately) with a cord that is connected to the Classic-Check. Sensormats are available for bed, chair, and wheelchair. See "Options" for available holsters and hangers.

**Power requirements:** Classic-Check is powered by plugging the integral power supply into a standard wall outlet (110VAC). Power is converted to 12VDC before entering the Classic-Check case (no line voltage in the monitor).

**Installation:** The Classic-Check is hung on the bed headboard or frame, or the chair/wheelchair back using the standard Velcro<sup>®</sup> strap or optional wire holster. (See "Options") It should be inaccessible to the monitored person. The appropriate Sensormat is installed per its Product Use Checklist on the product. See the Classic-Check User Guide for complete instructions.

**In-Service:** As with all Bed-Check Corporation products, complete personal in-servicing is available by a healthcare professional from your area Bed-Check Distributor.

**Features and Operation:** The Classic-Check has many standard features designed to facilitate the monitoring of fall risk patients.

- Monitoring begins automatically 5 seconds after patient weight is sensed on the Sensormat.
- Begin monitoring immediately by pushing RESET any time a patient is on the Sensormat.
- The alarm delay is 1, 2, or 3 seconds (settable) after weight is removed from the Sensormat. The Classic-Check sounds an audible alarm tone and signals the nurse call system.
- The RESET button puts the system on temporary hold for 25 seconds. At that time, if the patient is on the Sensormat, monitoring begins. If the patient is not on the Sensormat, permanent hold begins.
- Day and Nite modes. Setting to "Nite" illuminates the splash shield with a soft blue light and changes alarm volume to your Nite pre-set level.
- LEDs indicate HOLD and MONitor modes, and the 1, 2, or 3 second delay countdown.
- 7 volume settings each in Day and Nite modes.
- 4 tone and 6 tune settings.
- 10' power and nurse call systems cords.
- ETL listed. Conforms to UL Standard 1069, and to CSA Std C22.2 No. 205R.

**Options:**

- Holsters – Wire basket holders are available with 1" or 2" depth hangers.