

# POTTY-CHECK<sup>®</sup> IN-SERVICE REVIEW

## PROPER INSTALLATION OF THE POTTY-CHECK SYSTEM WHEN USED WITH A BASIC-CHECK<sup>™</sup> CONTROL UNIT

(Also consult the Basic-Check User Guide)



**A.** Remove Sensormat<sup>®</sup> from plastic container. Use the adhesive backing on the Sensormat to properly position it. It is usually placed between the two seat bumpers, but may be placed on the top edge of the toilet bowl. Locate it forward for lighter patients and rearward for heavier patients.



**B.** Plug the Potty-Check Sensormat's phone-style jack into the Basic-Check control unit.

*IMPORTANT: Use only Sensormats manufactured by Bed-Check Corporation with the Basic-Check control unit.*



**C.** Using either the integral hanger or the Velcro<sup>®</sup> backing, hang the Basic-Check on a bathroom handrail or wall where readily accessible to the caregiver, and out of reach of the patient.

Record date put in use on Sensormat cord tag.



**D.** Lower toilet seat and help patient/resident onto toilet. The control unit will arm itself automatically when the patient's/resident's weight is sensed. The alarm will beep one time, indicating that monitoring has begun.



**E.** If the patient/resident exits the toilet seat, the alarm will sound immediately. Alarm requires immediate intervention.



**F.** When an alarm has sounded, first assist the patient. The alarm may be silenced by seating the patient again, automatically putting Basic-Check into monitor mode. Otherwise silence the alarm by pushing the RESET button, putting Basic-Check into standby mode.



**G.** To remove the patient/resident from the toilet seat, allow the Basic-Check to alarm and press the RESET button on the Basic-Check one time. This puts the Basic-Check into standby mode.

### IMPORTANT:

Check for System Effectiveness Daily and/or at shift change.

Always check correct positioning of Sensormat after any alarm.

Use only Bed-Check products to maintain Bed-Check Warranty and to ensure System compatibility.

The physical characteristics, habits and factors that make each patient at risk for falls are different and may change daily. Please reassess as necessary.



**Bed-Check<sup>®</sup>**  
CORPORATION  
Reducing Falls Through Innovative  
Technology Since 1978

P.O. Box 170, Tulsa, Oklahoma 74101  
1-800-523-7956 • 1-918-592-3338  
Fax: 1-918-582-9828  
www.bedcheck.com